

Employee Name:	Date:	
Performance Criteria		
Education and competency validation methods: (VIP) Validation in Practice (DE) Demonstration simulation (v) Verbalization	Met	Not Met
Basics skills of using Temporal Scanner:		
• Attach disposable cap on probe		
• With probe flush on the center of forehead, depress red button, keep depressed		
• Slowly slide probe midline across forehead to the hair line, not down side of face.		
• Lift probe from forehead and touch on the neck just behind the ear lobe		
• Release the button, read, and record temperature		
• Temperature will remain on display for 30 seconds after the red button is released		
The Instrument:		
• Inspects the Probe lens for scratches and smudges. • Verbalizes lens should be shiny clean. • Cleanses lens with alcohol prep wipe when dirty • Cleanses lens with appropriate wipe after each use • Verbalizes circumstances when the device would need to be taken out of service.		
• Thermometer can be cleaned with any hospital-approved disinfectant, alcohol, or even bleach solutions		
The Battery:		
• Staff member is able to locate and interpret the bAtt indicator. • Verbalizes display image when battery is low • Seeks Manufacturer's Instructions for use for instructions for replacing battery.		
Any criteria initially not met, resulted in immediate remediation/reinstruction.		



**EXERGEN TEMPORAL THERMOMETER
COMPETENCY TOOL**

Evaluator signature: _____ Date: _____