

Point of Use Cleaning Observation Checklist

Requirement	Met	Not Met	Comments
Point of Use Cleaning in Procedural Areas and at the Bedside			
Ensure that there is a leak-proof, puncture-proof container with a biohazard affixed, in the procedure room to avoid staff injury			
Perform hand hygiene			
Don PPE as required per Instructions of Use for the cleaning agent being used			
Rinse/wipe the instruments with water to remove any visible blood, bioburden, etc.			
Place the instrument into the leak-proof, puncture-proof container			
Ensure that all of the instruments are in the open position so that enzymatic gets into all parts of the instrument			
Spray the instruments with Enzymatic			
Secure the container lid			
Remove PPE			
Perform hand hygiene			
Don gloves			
Transport the container to the Dirty Utility Room for pick up by the Sterile Processing Department			
Remove gloves			
Perform hand hygiene			
Point of Use Cleaning in the Operating Room			
Ensure that there is a leak-proof, puncture-proof container with a biohazard stick on it and/or case cart available to avoid staff injury			
Rinse /wipe the instruments with water to remove any visible blood, bioburden, etc.			
Place the instrument into the leak-proof, puncture-proof container (instrument container/casket) Note: A case cart with dirty instruments in it must have a biohazard sign on it			
Ensure that all of the instruments are in the open position so that enzymatic gets into all parts of the instrument			
Spray the instruments with Enzymatic			
Secure the container lid			
Transport the container to the case cart for pick up by Sterile Processing Department			
Remove gloves			
Perform hand hygiene			
Point of Use Cleaning in Endoscopy			
Ensure that there is a leak-proof, puncture-proof container with a biohazard affixed, in the Endoscopy Suite			
Wipe the exterior of the scope with water, to remove any visible			

Point of Use Cleaning Observation Checklist

Requirement	Met	Not Met	Comments
debris and other contaminates, followed by a wipe down with a sponge with detergent			
Suction and flush the channels through with water and detergent to remove any residue in the channel			
Remove buttons and discard (if disposable) or place in container for cleaning			
Brush the channel through until clean, and discard the brush			
Place the instrument into the leak-proof, puncture-proof container for transport			
Secure the container lid			
Transport the container to the scope cleaning room or case cart for Sterile Processing Department pick-up			
Remove gloves			
Perform hand hygiene			